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Important instructions for use for the nip® first moments manual breast pump

CONTENTS

1 //	General	14
2 //	Important information: before first use	14
3 //	Device description and safety instructions	14 – 15
	3.1 Intended use	
	3.2 Description	
	3.3 Potential discomfort or other symptoms	
	3.4 Hygiene	
	3.5 Information on bottle, teat and breast milk container	
4 //	Disassembly and assembly	15
	4.1 Description of the individual parts	
	4.2 Disassembly	
	4.3 Assembly	
5 //	Cleaning and disinfection	16
	5.1 Before first use	
	5.2 After every use and once per day	
6 //	Use	16
	6.1 Expressing breast milk	
	6.2 Feeding expressed breast milk	
7 //	Storage	17
8 //	Compatibility	17
9 //	Tips for expression	18
10 //	Troubleshooting	18
11 //	Disposal	18

1 // GENERAL

In the first months after birth, breast milk is the best food your baby can get. The composition is perfectly matched to the needs of your baby and strengthens the immune system. Every woman gets a different feeling when expressing milk. For this reason, the nip® first moments breast pump has two different efficiency levels that regulate the pump performance and thus make expression as comfortable as possible. The breast pump is moreover part of the nip® first moments All-in-One system which allows you to express the milk directly into the bottle and/or the breast milk container. For more information, please refer to point 8 "Compatibility". **We ask you to read the usage instructions carefully and to store them in a safe location for later use.**

2 // IMPORTANT INFORMATION: BEFORE FIRST USE

Before first use, disassemble the nip® first moments breast pump into all individual parts. Clean and disinfect the breast pump and the accessories before using them for the first time.

3 // DEVICE DESCRIPTION AND SAFETY INSTRUCTIONS

3.1 Intended use

The nip® first moments breast pump serves the expression of breast milk. Only use nip® first moments original accessories as this is the only way to ensure proper and fault-free functioning of the pump. Keep all components not in use out of the reach of children. Not a toy – individual parts pose a choking hazard.

3.2 Description

The nip® first moments breast pump is a manually operated milk pump with the following advantages:

- Works without electric power
- Easy cleaning as the pump can be disassembled into all individual parts
- Soft silicone cushion for a pleasant feeling during expression
- Collection of expressed breast milk

3.3 Potential discomfort or other symptoms

In case of illness of the breast (inflammation, open wounds) or unexpected reactions, only use the nip® first moments breast pump after consultation with your breastfeeding counsellor, midwife or your physician. If the breast milk is used for a premature baby, it is vital to observe the instructions of your physician.

3.4 Hygiene

The handling of breast milk requires a high level of hygiene to ensure healthy development of your child. Please observe point 5 "Cleaning and disinfection". The full suction performance can only be achieved with a carefully cleaned milk pump. Remove any body jewellery worn at the breast (e.g. piercings) as well as oil-based and fatty ointments and clean the breast with clear water before use. The breast pump is only intended for use by a female user.

3.5 Information about bottle, teat and breast milk container.

Bottle complies with the EN 14350 standard and is tested by independent labs. If you have any complaints, please send us the manual breast pump with packaging.

For your child's safety and health: WARNING!

- Always use this product with adult supervision.
 - Never use feeding teats as a soother.
 - Continuous and prolonged sucking of fluids will cause tooth decay.
 - Always check food temperature before feeding. Before using in the baby food warmer, please remove the screw ring and teat from the bottle.
 - Keep components not in use out of the reach of children.
 - Throw away at the first signs of damage or weakness.
 - Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
-
- Do not heat closed bottles filled with liquids or formula in the microwave – explosion risk.
 - Special care should be taken when heating up in the microwave, local overheating may occur. Always stir heated food and check the temperature before feeding. Breast milk may not be thawed in the microwave as valuable components are lost.
 - Breastfeeding is best for your child.

- Inspect the feeding teat and bottle before each use. Pull the feeding teat in all directions.
- Please replace bottles damaged from the inside (e.g. by cracks or scratches).
- Check regularly, that the teat hole and the anti-colic vent are not blocked.
- Do not use the teat for longer than 4 – 8 weeks, depending on how frequently it is used, for hygienic and safety reasons.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant (sterilizing solution) for longer than recommended, as this may weaken the teat.
- Not a toy - individual parts pose a choking hazard.
- Do not use the bottle to store food that is too hot. Protection against leaking cannot be assured if the bottle is used for carbonated or sparkling drinks.

4 // DISASSEMBLY AND ASSEMBLY

4.1 Description of the individual parts (drawing with numeration on page 3)

- 1 Hygiene cap
- 2 Breast cushion
- 3 Cylinder cap
- 4 Handle
- 5 Piston
- 6 Cylinder
- 7 Pump body
- 8 Vent
- 9 Adapter
- 10 Container
- 11 Lid
- 12 Screw ring
- 13 Teat
- 14 Cap

* is only necessary for the use of another nip® wide-neck bottle (not for the nip® first moments wide-neck bottle!)

4.2 Disassembly

Remove hygiene cap (1), breast cushion (2), cylinder cap (3) and vent (8) from the pump body (7). Afterwards, release the handle (4) from the fixing and from the piston (5) by slightly pushing it upwards. Finally, the cylinder (6) can be removed from the pump body (7) together with the piston (5). Release the piston (5) from the cylinder (6) by pulling strongly. Now, the breast pump should be disassembled into all individual parts and can be cleaned and disinfected.

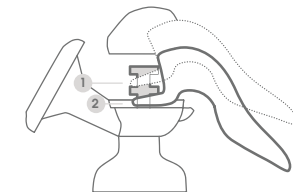
4.3 Assembly

From below, pull the piston (5) completely through the cylinder (6). Do not be afraid, this may possibly require a little bit more force. The piston (5) is correctly positioned when the broad side rests at the bottom of the cylinder (6). Insert the cylinder incl. piston (5+6) at the top into the pump body (7) and fasten the silicone edge of the cylinder (6) at the pump body (7).

Efficiency levels

Every woman gets a different feeling when expressing milk. For this reason, the nip® first moments breast pump has two different efficiency levels that regulate the pump performance and thus make expression as comfortable as possible.

- 1 **Level 1** (top position): For a weaker pump performance and stimulation
- 2 **Level 2** (bottom position): For a stronger pump performance

Hook the handle (4) into the piston (5) and press the upper part of the handle (4) towards the pump body (7). You will hear a soft click when it engages. Attach the cylinder cap (3) to the pump body (7). Attach the vent (8) to the opening at the lower end of the of the breast pump. The breast cushion (2) can be attached at the front of the pump body, if required. Expression is also possible without the breast cushion (2), depending on what feels better for you. Put the hygiene cap (1) onto the breast pump when the latter is not in use.

5 // CLEANING AND DISINFECTION

5.1 Before first use

- Before the first use, disassemble the breast pump, bottle and the accessories into all individual parts (point 4 "Disassembly and assembly") and sterilise everything in boiling water for 5 minutes; afterwards, remove the parts and let them cool down. When sterilising in boiling water, please always ensure there is a sufficient volume of liquid in the pot so that the articles are not damaged.

5.2 After every use and once per day

- After every use, disassemble the breast pump, bottle and the accessories into all individual parts (point 4 "Disassembly and assembly") and clean it using washing-up liquid.
- Do not use any aggressive cleaning agents or coarse / hard brushes / sponges for the cleaning. Do not use any aids such as brushes to clean the vent (8) as they might damage the vent.
- It is recommended to clean the parts directly after expression so that the breast milk does not dry up and the growth of bacteria is prevented.
- Alternatively, all parts except for parts made of silicone (breast cushion (2), cylinder (6), vent (8) and teat (13)) can be cleaned in the dishwasher. Small parts such as the piston (5) should only be cleaned in the cutlery basket of the dishwasher or by hand.
- When cleaned in the dishwasher, food residues may cause discolouration of the materials and material abrasion is possible.
- When using suitable disinfection devices or agents, please observe the manufacturer's instructions to prevent damage to the product.
- Following the cleaning, all parts are to be disinfected. You can sterilise the individual parts in boiling water for 5 minutes or disinfect them using a steriliser or microwave steriliser (please observe the manufacturer's instructions). When sterilising in boiling water, please always ensure there is a sufficient volume of liquid in the pot so that the articles are not damaged.
- After the disinfection, let all individual parts dry well before re-assembling them.
- Store all individual parts in a hygienic manner (clean and dry) in a place where they are protected from light.

6 // USE

6.1 Expressing breast milk

- Remove any body jewellery worn at the breast (e.g. piercing) as well as oil-based and fatty ointments and clean the breast with clear water before use.
- First of all, set the efficiency level (point 4 "Disassembly and assembly" / efficiency levels).
- Screw the container (10) to the pump ensuring a good seat and remove the hygiene cap (1) from the pump.
- Put the breast shield onto the breast. Make sure that the nipple is in the centre of the shield. This prevents the nipple from getting sore.
- Slightly press the shield against the breast so that no air can enter at the side and a vacuum is generated.
- Now start pumping using the handle (4). In addition to the efficiency levels, you can also influence the pump performance here by pressing the handle completely or only partially.

CAUTION!

- Only pump as much milk into the container so that the vent (8) does not get into contact with the collected milk in the container.
- If you plan to freeze the breast milk, do not fill the container over 150 mL as the milk will expand during freezing (risk of bursting!). For more information on the storage of breast milk, refer to point 7 "Storage".
- After the expression, put the pump onto a stable surface so that it does not fall over. If the pump falls over, the breast milk may run out.
- Screw the container off the pump and close it tightly using the supplied cover.
- Immediately after expression, clean all parts as described under point 5 "Cleaning and disinfection".

6.2 Feeding expressed breast milk

- Screw the container with the expressed breast milk off the pump.
- Mount the teat (13) on the screw ring (12) until it fits tightly.
- Screw the screw ring incl. the teat onto the bottle.

7 // STORAGE

The breast milk can be chilled, frozen, thawed and heated up in the breast milk container. To do this, please observe the following information when handling breast milk. To use the container as a bottle, you require a nip® first moments teat, screw ring and a cap. **Please observe the following information when handling breast milk:**

Breast milk	Storage location	Storage periods
Freshly expressed breast milk	Room temperature	6-8 hours
	Fridge (not in the door)	72 hours at 4 °C
	Freezer	6 months at -18 °C
Thawed breast milk (unopened)	Fridge (not in the door)	24 hours at 4 °C
Thawed breast milk (opened)	Fridge (not in the door)	12 hours at 4 °C

Thawing the breast milk:

You can leave the breast milk to thaw carefully in the freezer or at room temperature. If you allow the breast milk to thaw at room temperature, please feed directly afterwards. If you have to thaw the milk quickly, you can do this in a lukewarm water bath (max. 37 °C). In this case, please feed directly afterwards.

CAUTION: Breast milk may not be thawed in the microwave as valuable components are lost.

Important information:

- Do not refreeze thawed, heated or unused breast milk. Dispose of this.
- Shake the container lightly before feeding to mix the heat, as well as the breast milk and milk fat.
- If you express breast milk several times within 24 hours, the freshly expressed breast milk can be poured into already chilled breast milk. Please allow the fresh breast milk to cool first and only pour them together once it has reached the same temperature as the chilled breast milk (cold on cold).
- When transporting the breast milk, please ensure that the chill chain is adhered to.
- Only heat the breast milk container when it is open (risk of explosion)!
- You can ask your breastfeeding counsellor, midwife, doctor or pharmacist for precise instructions about long-term storage.

Use the nip® baby food warmer to heat the breast milk to the correct drinking temperature of approx. 37 °C. Before using in the baby food warmer, please remove the screw ring and teat from the bottle. Always check the temperature of the foodstuff before feeding.

8 // COMPATIBILITY

The nip® manual milk pump is part of the nip® first moments All-in-One system.

The All-in-One system allows you to directly pump the breast milk into the container without an adapter and to store it safely. To feed your child the breast milk, you need only to remove the cover, heat the breast milk (point 7 "Storage") and screw on the screw ring and the naturally formed teat.

In addition, every hand milk pump includes a white adapter (9). This adapter allows you to pump milk directly into a nip® wide-neck bottle if you do not have any nip® first moments wide-neck bottles.

9 // TIPS FOR EXPRESSION

If the expression of breast milk is new to you, you will need some time and practice until it will work properly. Do not get discouraged if it does not work smoothly right from the beginning. It is, however, still important to observe the following points:

Hygiene

The handling of breast milk requires special attention to be paid to hygiene. For this reason, you should clean the breast using clear water before the expression. Clean and/or disinfect the pump as specified under point 4 "Cleaning and disinfection" and wash your hands.

Preparation

Take your time for the expression. Sit down at a quiet place where you are undisturbed and prepare yourself something to drink, read or have the TV remote control ready to hand. Ensure that you sit comfortably. You can best prepare the breast for the coming milk expression by massaging or warming the breast before the expression.

Expressing

At the beginning, the breast milk has a different composition than after some minutes of expression. Emptying both breasts completely ensures that the perfect milk quantity will be reproduced and inflammation or breast engorgement will be prevented.

During nursing, the let-down reflex is automatically triggered by the close contact with your baby. During expression, this is not always easy. It may therefore be helpful if your baby is near you or if you look at a picture of him/her. Sometimes also the idea and anticipation of holding your baby in your arms again soon may be helpful.

If you express and nurse at the same time, make sure to put the baby to the breast first and express afterwards.

The expression period should lie between 20 and 30 minutes. To support the oxytocin release to the maximum extent, you should express your breasts alternately:

First express both breasts for 7 minutes one after the other, then for 5 minutes and finally for 3 minutes.

Do not ignore discomfort or other symptoms

If you still experience discomfort or other symptoms during the expression, do not ignore them and contact your breastfeeding counsellor, midwife, your physician or pharmacist.

10 // TROUBLESHOOTING

There is no pump suction at all or the suction is too weak

- First of all, check whether the pump has been correctly assembled. Specifically ensure that the piston (5) has been completely pulled through the cylinder (6) and the cylinder is firmly attached to the pump body
- Check whether the vent (8) has been firmly attached to the bottom of the pump body
- Ensure during expression that the breast completely seals the shield so that no air can enter at the sides.

No milk is expressed

- Make sure that the milk pump has been assembled correctly.
- Look for a quiet place, relax and try again after a few minutes.
- If you are still unable to express milk, contact your breastfeeding counsellor, midwife, your physician or pharmacist.

11 // DISPOSAL

The nip® first moments breast pump is made of polypropylene (PP) and silicone. Dispose of it complying with the local regulations.

If you need more information, please visit us online at firstmoments.nip.family

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